

User Guide



Changing the Way Anesthesia is Delivered.



Wiley Spinal™ Insertion Instructions



1: Loss of Resistance

- Obtain Loss of Resistance using Tuohy Peel-Away and LOR syringe.
- Remove needle from peel-away



2: Placement

- Align black triangles in a cephalad direction.
- Insert and advance Wiley Spinal until dural click is detected.
- Remove grey capped stylet to confirm CSF feedback.



3: Advancement

- Rotate clear needle hub counterclockwise 360°.
- Stabilize clear needle hub and advance flexible cannula to peel-away hub.



4: Peel-Away Removal

- Remove peel-away and continue to advance flexible cannula off needle into patient.
- Remove needle.



5: Insertion

- Immediately insert Wik-Wire and tightly secure luer-lock connection.
- Remove cap at proximal end of extension set and confirm passive filling.



6: Securement

- Place hub support pad over insertion site and apply dressing.
- Titrate medication.



141 Sal Landrio Dr., Crossroads Business Park
Johnstown, NY 12095
800.866.3342
www.wileyspinal.com